

## **Gynecologic Pathology Grossing Guidelines**

**Specimen Type:** CERVICAL BIOPSY

**Gross Template:**

Labeled with the patient's name (\*\*\*) , medical record number (\*\*\*) , designated "\*\*\*\*" , and received [*fresh/in formalin*] are multiple [*color, consistency*] portions of tissue measuring \*\*\* x \*\*\* x \*\*\* cm in aggregate and ranging from \*\*\* cm to \*\*\* cm in greatest dimension. The specimen is entirely submitted in [*describe cassette summary*].

**Cassette Submission:** All tissue submitted

- If necessary section of cervix is taken parallel to the axis of the cervical canal to include squamo-columnar junction. Submit entire specimen.
- If specimens are labeled with specific identification (e.g., anterior lip, posterior lip), submit separately.