

Gynecologic Pathology Grossing Guidelines

Specimen Type: ENDOCERVICAL CURRETTINGS (ECC)

Gross Template:

Labeled with the patient's name (***) , medical record number (***) , designated "****" , and received [*fresh/in formalin*] is a *** x *** x *** cm aggregate of minute red-tan tissue fragments admixed with blood clot. The specimen is entirely submitted [*describe cassette summary*].

Cassette Submission: All tissue submitted

- Note: If received on Teflon or gauze, carefully scrape with a CLEAN blade onto tissue wrap. If free floating in formalin, pour through a nylon biopsy bag