

Ophthalmic Pathology Grossing Guidelines

Specimen Type: PTERYGIUM

Gross Template:

Labeled with the patient's name (**), medical record number (**), designated "****", and received [*fresh/in formalin*] is an unoriented [*pink-tan*] soft tissue fragment measuring ** x ** x ** cm [*If any, indicate provided orientation*]. The specimen is entirely submitted in [*describe cassette submission*].

INK KEY:

Green- resection margin

Note: deep surface is often the surface adherent to the cardboard & it may appear dull in comparison to the superficial surface

Cassette Submission: 1 cassette

- Submit entirely, unsectioned