

DERMATOPATHOLOGY GROSSING GUIDELINES

RUSH CASES:

For all RUSH cases, the dermpath fellows AND the skin service resident should be notified via email about the case.

Specimen Type: SKIN CURRETING

Gross Template:

Labeled with the patient's name (**), medical record number (**), designated "****", and received [*fresh/in formalin*] are multiple fragments of [*color*] skin measuring ** x ** x ** cm in aggregate ranging from ** to ** cm in greatest dimension. The specimen is entirely submitted in [*describe cassette submission*].

Cassette Submission: All tissue submitted