

Gynecologic Pathology Grossing Guidelines

Specimen Type: TUBAL LIGATION

Gross Template:

Labeled with the patient's name (***), medical record number (***), designated "****", and received [*fresh/in formalin*] is a segment of fallopian tube measuring *** cm in length x *** cm in diameter. Fimbriae are [*present/absent*]. The lumen is patent and measures up to *** cm in diameter. No lesions are identified. Representative sections are submitted [*describe cassette submission*].

Cassette Submission: 1 cassette

- Submit at least 2 complete cross sections, if able
- If you do not identify a lumen, submit the entire specimen
- **If the specimen does not appear to be fallopian tube (i.e. blood vessel or round ligament) verify attending pathologist immediately.**