

Gynecologic Pathology Grossing Guidelines

Specimen Type: MORCELLATED LAPRASCOPIC HYSTERECTOMY

Gross Template:

Labeled with the patient's name (**), medical record number (**), designated "****", and received [*fresh/in formalin*] is a ** gram morcellated [*total/supracervical hysterectomy*] received in multiple portions. The portions range from ** to ** cm in maximum dimension and amount in aggregate to ** x ** x ** cm . The [*cervix/endocervical stump*] measures ** cm in length x ** cm in diameter. There is a ** cm average endometrial thickness. Portions with endometrium and serosa have a ** cm average myometrial thickness.

Identifiable serosa is [*pink, smooth, glistening*]. Identifiable endometrium is [*red and thickened, yellow and atrophic, denuded*]. The myometrium is [*pink-tan, trabeculated, remarkable for cystic spaces (adenomyosis), leiomyomas-number, size, % the fibroids account for of the total specimen, hemorrhage, necrosis, calcification, location*]. unremarkable/remarkable for leiomyomata]. No lesions or masses are identified. Representative sections are submitted [*describe cassette submission*].

Cassette Submission: 7-8 cassettes

- 3 cassettes of leiomyoma if grossly unremarkable
 - o You can add 3 pieces in 1 cassette
- 2 cassettes of endometrium/myometrium/serosa
- Anterior and posterior cervix/upper endocervical stump, perpendicular