

DERMATOPATHOLOGY GROSSING GUIDELINES

RUSH CASES:

For all RUSH cases, the dermpath fellows AND the skin service resident should be notified via email about the case.

Specimen Type: FORESKIN

Gross Template:

MMODAL COMMAND: INSERT FORESKIN

It consists of an unoriented skin excision measuring [*measure in three dimensions****]. The specimen is serially sectioned to reveal [*describe any lesions/pink-tan cut surfaces****]. [*The specimen is entirely submitted/Representative sections are submitted****] in cassette [***].

Cassette Submission:

- Newborns – no tissue submitted (gross only)
 - Submit one cassette if gross abnormality identified
- Adults – one cassette of representative tissue
 - Submit 1-2 additional cassettes if gross abnormality identified