

## Hematopathology Grossing Guidelines

**Specimen Type:** RESECTION (tumor staging)

**Gross Template:**

**MMODAL COMMAND:** "INSERT LYMPH NODE DISSECTION"

It consists of a [*measure in three dimensions\*\*\**] cm portion of tan-yellow fibroadipose tissue. Sectioning reveals [*describe # of lymph nodes found\*\*\**] lymph node candidates, ranging from [*smallest to largest\*\*\**] cm in maximum dimensions. [*The lymph node is/The lymph nodes are\*\*\**] sectioned to reveal [*describe cut surface and comment on grossly positive lymph nodes, measuring the grossly positive tumor\*\*\**]. The specimen is entirely submitted. [*cassette submission/ if lymph nodes are serially sectioned, only place one lymph node in each cassette or you MUST differentially ink them\*\*\**] [*Describe salivary gland if present-only submit one section of gland if present\*\*\**]

[*insert cassette summary\*\*\**]

**Cassette Submission:** Submit all lymph nodes identified