CARDIOVASCULAR PATHOLOGY GROSSING GUIDELINES

<u>NOTE:</u> Please review all complex or "interesting" gross specimens with Drs. Michael Fishbein, Gregory Fishbein, or Ryan Lau.

Specimen Type: TEMPORAL ARTERY BIOPSY

Procedure:

- 1. Measure length and diameter.
- 2. DO NOT CUT SPECIMEN. Histology will cut at the time of embedding

 *** Arteries biopsied to rule out vasculitis should be grossed by the
 histotechs per protocol, not by PAs or residents.

Gross Template:

MMODAL COMMAND: INSERT TEMPORAL ARTERY

It consists of a segment of vessel measuring [***] cm in length x [***] cm in diameter. The specimen is entirely submitted, to be sectioned by histology, in cassette [describe cassette submission***].

Cassette Submission: 1 cassette

- Do NOT cut specimen
- Leave a case flag for histology