GASTROINTESTINAL PATHOLOGY GROSSING GUIDELINES

Specimen Type: SLEEVE GASTRECTOMY

Procedure:

- 1. Measure the length, range of diameter or circumference, and wall thickness of resected portion of stomach.
- 2. Describe the appearance of serosa and mucosa

Gross Template:

MMODAL COMMAND: "INSERT SLEEVE"

It consists of an [intact, disrupted***] sleeve gastrectomy measuring [***] cm in length, [***] cm in open circumference. There is a [***] cm in length staple line at the resection margin. The perigastric fibroadipose tissue extends up to [***] cm from the gastric wall.

The serosal surface is [pink-tan and unremarkable/ remarkable for describe adhesions, plaques, full-thickness defects (perforations or enterotomies)***]. The mucosal surface is [brown-tan and unremarkable/ remarkable for [describe ulcers/erosions/polyps/loss of folds/nodularity ***]. No lesions are grossly identified. [Describe lesions/polyps- if present.***] Representative sections are submitted in cassette [***].

Cassette Submission: 1 cassette (additional cassette(s) if necessary to demonstrate pathology)

- Submit two representative sections of stomach wall in one cassette
 - o Include area of congestion or any other abnormal findings
- Submit all polyps entirely