

BONE AND SOFT TISSUE PATHOLOGY GROSSING GUIDELINES

- NOTE: Document in your cassette summary which cassettes (for each specimen part) are submitted for decal
 - Ex: Representative sections are submitted (A1, A3; B2-B4; C3-C4 following decalcification).
 - Ex: Representative sections are submitted:
 - A1- bone (decal)
 - A2- skin resection margin, perpendicular
 - B2- tibia shave (decal)

MModal Command: "INSERT DISC"

Specimen Type: INTERVERTEBRAL DISC

Gross Template:

It consists of multiple white-tan soft tissue fragments measuring [*measure in three dimensions****] cm in aggregate. [*The specimen is entirely submitted/Representative sections are submitted****].

Cassette Submission: 1 cassette

- Submit more cassettes if lesion/mass is identified
- Decalcify if necessary