

Gynecologic Pathology Grossing Guidelines

Specimen Type: CERVICAL BIOPSY

Gross Template:

MMODAL COMMAND: "INSERT CERVIX"

The specimen is received in one formalin filled container, labeled with the patient's name (*[last name, first name***]*), medical record number (*[insert MRN***]*), and designated as "*[Dictate full description listed in Beaker. Ensure that the specimen label matches the Beaker order***]*". It consists of multiple pink-tan portions of soft tissue measuring *[measure in three dimensions***]* cm in aggregate. The specimen is entirely submitted, in a mesh bag, in *[describe cassette submission***]*.

*[Insert grosser's initials and today's date (SM 01/01/2000)***]*

Cassette Submission: All tissue submitted

- If necessary section of cervix is taken parallel to the axis of the cervical canal to include squamo-columnar junction. Submit entire specimen.
- If specimens are labeled with specific identification (e.g., anterior lip, posterior lip), submit separately.