

Ophthalmic Pathology Grossing Guidelines

Specimen Type: PTERYGIUM

- FOR ALL JS SPECIMENS LEAVE FORMALIN IN CONTAINER AND KEEP CONTAINER UNTIL AFTER SIGN OUT. DO NOT FILTER THESE SPECIMENS THROUGH A MESH BAG.
- Always ink the deep surface of these specimens.

Gross Template:

MMODAL Command: "INSERT PTERYGIUM"

It consists of a [*measure in three dimensions****] cm pink-tan soft tissue fragment. The specimen is entirely submitted, in a mesh bag, in cassette [*describe cassette submission****].

INK KEY:

Green resection margin

Note: deep surface is often the surface adherent to the cardboard & it may appear dull in comparison to the superficial surface

Cassette Submission: 1 cassette

- Submit entirely, unsectioned