

Genitourinary Grossing Guidelines

Specimen Type: TESTIS BIOPSY

Comment: These are performed either for tumor or for analysis of spermatogenesis. They may be processed identically. It is important to treat these biopsies **GENTLY**, as testicular parenchyma is very delicate.

Gross Template:

MMODAL Command: "INSERT TESTICLE BIOPSY"

It consists of [*state number of cores***] brown-tan soft tissue fragments measuring [*measure in three dimensions***] cm in aggregate. The specimen is entirely submitted in cassette [****].

Cassette Submission: All tissue submitted