

Gynecologic Pathology Grossing Guidelines

Specimen Type: ECTOPIC/TUBAL PREGNANCY

Gross Template:

Labeled with the patient's name (***), medical record number (***), designated "****", and received [*fresh/in formalin*] is a [*disrupted/intact*] salpingectomy measuring *** cm in length x *** cm in diameter. Fimbriae are [*present/absent*]. The lumen is patent and ranges from *** to *** cm in diameter. There is a [*describe rupture- (measure length and width), note associated hemorrhage, blood clot*]. The lumen contains [*blood, embryo, chorionic villi, other-weigh and measure if appropriate*]. Representative sections are submitted [*describe cassette submission*].

Cassette Submission: 3-4 cassettes

- Submit cross sections to demonstrate site of rupture, perpendicular sections
- Submit cross sections to demonstrate uninvolved fallopian tube
- **If no gestational sac is grossly identified, submit the entire fallopian tube sequentially**