

## **Gynecologic Pathology Grossing Guidelines**

**Specimen Type:** MYOMECTOMY (morcellated/laparoscopic)

**Gross Template:**

Labeled with the patient's name (\*\*\*), medical record number (\*\*\*), designated "\*\*\*\*", and received [*fresh/in formalin*] is a \*\*\* gram, \*\*\* x \*\*\* x \*\*\* cm aggregate of pink-tan, semi-firm irregularly shaped tissue fragments. The fragments range from \*\*\*-\*\*\* cm in maximum dimension. There are no areas of hemorrhage, necrosis, or calcification. Representative sections are submitted [*describe cassette submission*].

**Cassette Submission:** 3 cassettes

- Focus on yellow, calcified, or hemorrhagic areas
- If small enough-entirely embed