Genitourinary Grossing Guidelines

Specimen Type: TRANSURETHRAL RESECTION OF BLADDER TUMOR (TURBT)

Procedure:

- 1. Note overall dimensions of specimen; describe the texture color of tissue.
- 2. Submit all tissue.

Gross Template:

Labeled with the patient's name (***), medical record number (***), designated "***", and received [fresh/in formalin] is a *** gram, *** x *** x *** cm aggregate of semi-firm, pinktan portions of tissue. The portions range from *** to *** cm in greatest dimension. All portions appear homogeneous in color and texture. There [are/are no] areas of hemorrhage, degeneration, or calcification. The specimen is entirely submitted [describe cassette submission].

<u>Cassette Submission:</u> All tissue submitted if \leq 20 cassettes. If > 20 cassettes, please check with attending staff.