

Gynecologic Pathology Grossing Guidelines

Specimen Type: ENDOCERVICAL CURRETTINGS (ECC)

Gross Template:

MMODAL COMMAND: "INSERT ECC"

The specimen is received in one formalin filled container, labeled with the patient's name ([*last name, first name****]), medical record number ([insert MRN***]), and designated as "[*Dictate full description listed in Beaker. Ensure that the specimen label matches the Beaker order****]". It consists of a [*measure in three dimensions****] cm aggregate of minute red-tan tissue fragments with mucoid material. The specimen is entirely submitted, in a mesh bag, in [*describe cassette submission****].

[*Insert grosser's initials and today's date (SM 01/01/2000)****]

Cassette Submission: All tissue submitted

- Note: If received on Teflon or gauze, carefully scrape with a CLEAN blade onto tissue wrap. If free floating in formalin, pour through a nylon biopsy bag