

DERMATOPATHOLOGY GROSSING GUIDELINES

RUSH CASES:

For all RUSH cases, the dermpath fellows AND the skin service resident should be notified via email about the case.

Specimen Type: SKIN CURRETING

Gross Template:

MMODAL COMMAND: "INSERT SKIN CURRETING"

It consists of multiple fragments of [*color/tan-brown****] skin measuring [*measure in three dimensions****] cm in aggregate ranging from [*smallest to largest****] cm in greatest dimension. The specimen is entirely submitted in [*describe cassette submission****].

Cassette Submission: All tissue submitted