BONE AND SOFT TISSUE PATHOLOGY GROSSING GUIDELINES

- NOTE: Document in your cassette summary which cassettes (for each specimen part) are submitted for decal
 - Ex: Representative sections are submitted (A1, A3; B2-B4; C3-C4 following decalcification).
 - Ex: Representative sections are submitted:
 - A1- bone (decal)
 - A2- skin resection margin, perpendicular
 - B2- tibia shave (decal)

MModal Command: "INSERT SYNOVIUM"

Specimen Type: SYNOVIUM

Gross Template:

It consists of multiple fragments of pink-tan, soft tissue ranging from [smallest to largest***] cm in maximum dimension, and measuring [measure in three dimensions***] cm in aggregate. [The specimen is entirely submitted/Representative sections are submitted in cassette***].

Cassette Submission: 1 cassette

Submit more cassettes if lesion/mass is identified