# Hepatobiliary Pathology Grossing Guidelines

## **Specimen Type:** WEDGE BIOPSY

## Gross Template:

## MMODAL COMMAND: "INSERT LIVER WEDGE"

It consists of a partial hepatectomy weighing [\*\*\*] grams, and measuring [*measure in three dimensions*\*\*\*] cm. The capsule is [*red-tan, smooth, diffusely nodular*\*\*\*]. The hepatic resection surface measures [\*\*\* x \*\*\*] cm.

The specimen is serially sectioned to reveal a [*measure mass in three dimensions, describe color, shape, and consistency*\*\*\*]. The mass is located [\*\*\*] cm from the capsule, and [\*\*\*] cm from the parenchymal resection margin. The remaining uninvolved parenchyma is [*firm, red-tan, green-tan, homogeneous, fibrotic, nodular* \*\*\*]. The specimen is entirely submitted in cassettes [\*\*\*].

#### INK KEY:

Black parenchymal resection margin Blue capsule overlying tumor

[insert cassette summary]

### Cassette Submission:

All tissue submitted

- Submit entirely
- Add the appropriate histology order